

The ComfortSampler® device is a safe and simple solution.

The ComfortSampler's innovative design and function allow you to provide the best in patient care at an affordable price. The fast, single step procedure fills automatically with minimal arterial pressure and offers superior sample stability. The small gauge needle reduces puncture pain and hematomas, which is of paramount importance to the patient. The integrated needle protector provides the ultimate protection from accidental puncture wounds.


Therapists benefit from ComfortSampler's:

- ▶ Accurate results
- ▶ Easy-to-handle design
- ▶ Sample stability
- ▶ Superior safety and performance

Patients benefit from ComfortSampler's:

- ▶ Quick procedure
- ▶ Small sample volume
- ▶ Less traumatic and painful puncture

Blood gas collection made easy.

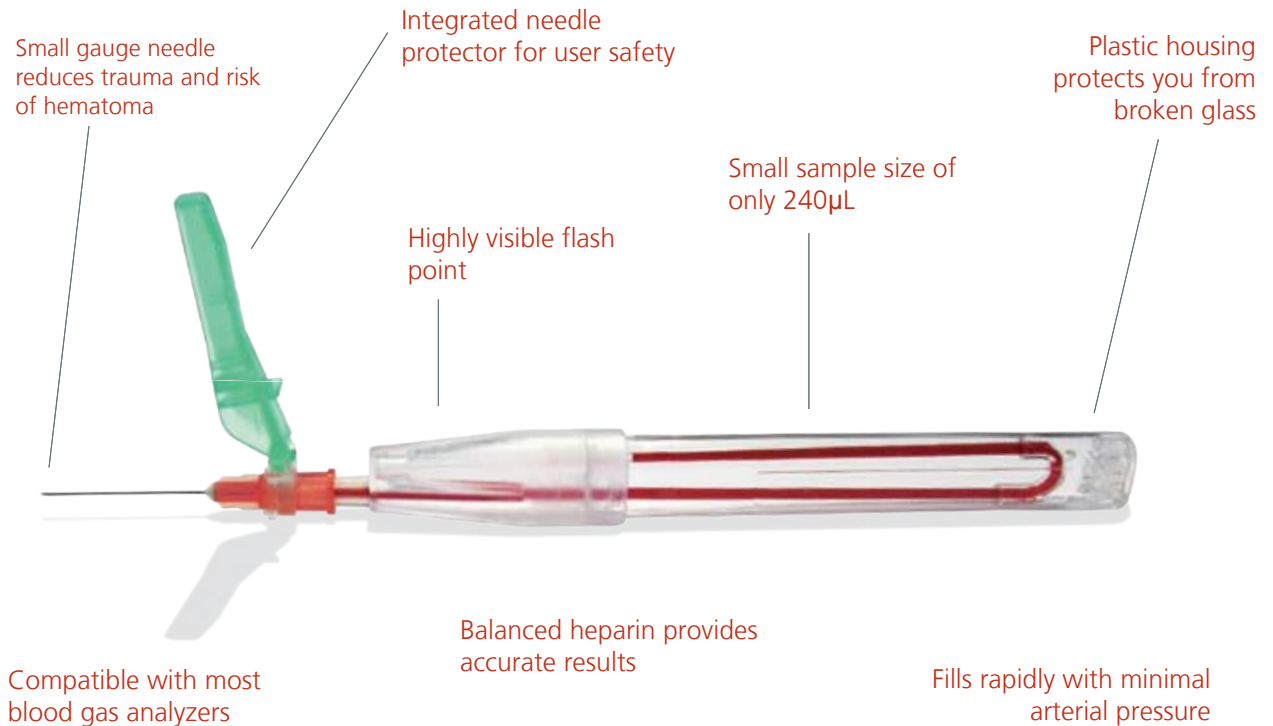

1. Place needle onto ComfortSampler.



2. Prepare puncture site.



3. Perform arterial puncture. Allow capillary to fill.



4. Close needle protector by pressing against a firm surface.

Safe for you....



...and the lives you touch.

Exceptional Safety

The ComfortSampler device has glass capillaries housed in a plastic shell which protects the therapist from broken glass. The SurGuard2™ safety needle protects against accidental puncture, allows for safe and easy disposal, and follows OSHA safety guidelines.

High Visibility Flash

The transparent construction of the ComfortSampler device makes it easy to identify when the artery has been punctured.

Increased Sample Stability

The glass capillary tubes keep patient samples stable and cool the samples quickly. Samples do not require icing if tested within 30 minutes.

Convenient Packaging

The ComfortSampler device comes in multiple package configurations. Individually wrapped for convenience, kits contain the accessories needed to perform the sample collection. Bulk packaging options are also available to meet high volume needs.

Ordering Information

Part No.	Description	Qty.
BP0600	ComfortSampler with Accessories Each kit contains: 1 ComfortSampler device 1 25-gauge x 1" (25 mm) needle with needle protector (sterile) 1 protective cap 1 2" x 2" (5 x 5 cm) gauze sponge (sterile) 1 alcohol prep pad (sterile) 1 specimen bag	50/Box
BP0610	ComfortSampler Basic Kit Each kit contains: 1 ComfortSampler device 1 25-gauge x 1" (25 mm) needle 1 protective cap	50/Box
BP0620	ComfortSampler Bulk Each box contains: 300 ComfortSamplers 300 25-gauge x 1" (25 mm) needle	300/Box
BP0630	ComfortSampler Each box contains: 300 ComfortSamplers	300/Box
BP0640	ComfortSampler Bulk Each box contains: 300 ComfortSamplers 300 25-gauge x 1" (25 mm) needle with needle protector	300/Box

For assistance in ordering a supply of OPTI Medical ComfortSampler products, or to have your local sales representative contact you, please email info@optimedical.com or call (800) 490-6784 or (770) 510-4444.

OPTI Medical Systems, Inc.
235 Hembree Park Drive
Roswell, GA 30076
www.optimedical.com

Ask about the portable OPTI® CCA and OPTI® R blood gas analyzers for your blood gas analysis needs.

CAUTION: Handle and dispose of all biologic samples and blood collection needles and equipment in accordance with your facility's policies. Obtain appropriate medical attention in the event of any exposure to biologic samples (e.g., through a puncture injury) since samples may transmit infectious diseases. Use the products only as directed in the product insert.



© 2007 OPTI Medical Systems, Inc. All rights reserved.
OPTI and ComfortSampler are registered trademarks of OPTI Medical Systems, Inc.
SurGuard2 is a trademark of Terumo Medical Corporation
PM0016 Rev. C



ComfortSampler® Arterial Blood Gas Collection Kits

Safe for you....



...and the lives you touch.